



## Liver<sup>7</sup> IgG

Order Code: LI7DIV-24

BlueDiver Protocol: 02

### 1. INTENDED USE

BlueDiver Dot Liver<sup>7</sup> IgG is an Immunodot kit intended for the detection, in human sera only, of IgG autoantibodies against the following antigens : M2/nPDC, gp210, sp100, LKM1, LC1, SLA and F-actin.

This kit is intended to confirm results of patterns obtained by immunofluorescence, the screening and reference method in autoimmunity; the kit is intended as an aid in the diagnosis of several autoimmune diseases (for more details, see 11.5 *Auto-antibodies diagnostic values*).

The test is intended for a large, routine population. This kit is strictly reserved for professional use in clinical analysis laboratories. This kit is strictly foreseen as an automated test and can only be used in a BlueDiver Instrument Model I or II (hereinafter named BDI I or BDI II respectively).

For a semi-quantification of the test results, it is necessary to use the BlueScan scanner system and the Dr Dot software. This system is not included in the BDI I, but is included in the BDI II (see point 4).

### 2. PRINCIPLE OF THE TEST

This kit and all its components are intended to be performed exclusively on the BDI I or II.

The test is based on the principle of an Enzyme Immunoassay. The strips are composed of a membrane fixed on a specific plastic support. During the automated test procedure, the BDI sequentially incubates the strips in the wells of ready-to-use reagent cartridges. Briefly: the strips are first incubated with diluted patients' sera. Human antibodies, if present, bind to the corresponding specific antigen(s) on the membrane. Unbound or excess antibodies are removed by washing. Upon further incubation into AP-conjugated goat antibodies against human IgG, the enzyme conjugate binds to the antigen-antibody complexes. After removal of excess conjugate by washing, the strips are finally incubated into a substrate solution. Enzyme activity, if present, leads to the development of purple dots on the membrane pads. The intensity of the coloration is directly proportional to the amount of antibody present in the sample.

The kit is composed of 24 single-use tests.

### 3. KIT CONTENTS

**Prior to any use of the kit, please check that all the items listed are present. Please also check if the characteristics of the product are corresponding to those described hereafter.**

**If one of the items is missing or damaged, please do not use the kit and contact your distributor.**

#### 3.1 COMPONENTS

Strips	3 x 8 units on plastic supports breakable individually; sealed in aluminium pouch. Each strip is for single-use.		CARTRIDGE
<p>9 dots on each: 1 positive control (RC) 7 antigens 1 negative control (CO)</p>	<p>24 units having each 7 compartments; sealed.</p> <p>Diluent buffer I<sup>st</sup> position, 1 x 1,4 ml (yellow) <i>contains</i> H<sub>2</sub>O • TBS • NaCl • Tween • BSA • Preservatives • dye • antifoam emulsion</p> <p>Wash buffer II<sup>nd</sup>, III<sup>rd</sup>, IV<sup>th</sup> and VI<sup>th</sup> position, 4 x 1,4 ml (colourless) <i>contains</i> H<sub>2</sub>O • TBS • NaCl • Tween • Preservatives • antifoam emulsion</p> <p>Conjugate V<sup>th</sup> position, 1 x 1,4 ml (red) <i>contains</i> H<sub>2</sub>O • TBS • NaCl • KCl • MgCl<sub>2</sub> • AP-conjugated goat anti-human IgG • Preservatives • dye • antifoam emulsion</p> <p>Substrate VII<sup>th</sup> position, 1 x 1,4 ml (pale yellow solution) <i>contains</i> H<sub>2</sub>O • Preservatives • MgCl<sub>2</sub> • TBS • NBT stabilizer • NBT • BCIP</p>		<p><b>Documents:</b> Instructions for Use (IFU), Certificate of Analysis (CoA)</p>

Abbreviations in alphabetic order:

AP = Alkaline Phosphatase; BCIP = Bromo-Chloro-Indolyl-Phosphate; BSA = Bovine Serum Albumin; KCl = Potassium Chloride; MgCl<sub>2</sub> = Magnesium Chloride; NaCl = Sodium Chloride; NBT = NitroBlue Tetrazolium; TBS = Tris Buffer Saline

For more information on the composition and concentration of the active ingredients used, please refer to the MSDS available on request or on [www.d-tec.be](http://www.d-tec.be).

### Symbols used on kit labels

	Attention : consult instructions for use Attenzione : consulti le istruzioni per uso Achtung :Gebrauchsanwendung beachten Attention : consulter le mode d'emploi Atención : consultar las instrucciones Atenção : consultar instruções para uso Προσοχή : Συμβουλευτείτε τις οδηγίες χρήσης	<b>STRIP</b>	Coated strip Strips rivestita Streifen Bandelette Tira Tira Στιγμάτων
<b>IVD</b>	In vitro diagnostic medical device Dispositivo medico diagnostico in vitro Zur medizinischen diagnostischen Anwendung in vitro Dispositif médical de diagnostic in vitro Dispositivo médico para uso diagnostico in vitro Dispositivo médico para uso diagnostico in vitro Ιατρικό υλικό για διάγνωση In Vitro		CE Mark Marcatura CE CE-Kennzeichnung Marquage CE Marca CE Marcação CE μονογράφιση CE
	To be stored from 2°C to 8°C Conservazione da 2 – 8°C bei 2°C bis 8°C lagern A conserver de 2°C à 8°C Almacenar a 2 - 8°C Armazenar a 2 – 8°C Αποθηκεύστε στους 2 έως 8°C		For ... uses Per ... dosaggi Für ... Anwendungen Pour ... utilisations Para ... usos Para ... utilização για ... χρήσεις
<b>LOT</b>	Batch Number Lotto numero Chargennummer Désignation du lot Denominacion de lote Número do lote Κωδικός	<b>REF</b>	Code Codice Artikelnummer Référence Código Código Κωδικός
	Use by (last day of the month) Utilizzare prima del (ultimo giorno del mese) Verwendbar bis (letzter Tag des Monats) Utiliser avant (dernier jour du mois indiqué) Estable hasta (usar antes de ultimo dia del mes) Data limite para utilização (ultimo dia do mês) Χρήση έως (τελευταία ημέρα του μήνα)		To be protected from direct sunlight Proteggere dalla luce Vor Licht schützen Protéger de la lumière Proteja de la luz Proteger da exposição à luz Προστατεύετε τον αντιδραστήριο
<b>CART</b>	Cartridge Cartuccia Patrone Cartouche cartucho cartucho κασέτα		Manufactured by Fabricado da Hergestellt von Fabriqué par Fabricado por Fabricado por Κατασκευάζεται από την

### 3.2 Antigens used

M2/nPDC	E1, E2, E3 subunits of Pyruvate Dehydrogenase Complex (purified from bovine heart)
gp210	Glycoprotein of the nuclear pore complex (36-amino acid sequence corresponding to the C-terminal cytoplasmic tail of gp210, human, recombinant, expressed in E.coli)
sp100	100 kD protein of the nuclear body (recombinant, human, expressed in Baculovirus-infected Sf9 cells)
LKM1	Cytochrome oxydase P450 2D6 (liver-kidney microsome type I antigen), Full length (recombinant, human, expressed in Baculovirus-infected Sf9 cells)
LC1	Formiminotransferase cyclodeaminase (liver cytosol type I antigen) (recombinant, human, expressed in Baculovirus-infected Sf9 cells)
SLA	Soluble Liver Antigen (recombinant, human, expressed in E.coli bacterial cells)
F-actin	In-vitro polymerized actin filaments (prepared from purified G-actin (rabbit skeletal muscle))

## 4. MATERIAL REQUIRED BUT NOT PROVIDED

### BDI I:



The BDI I is an instrument that performs the various steps of incubation and washing of D-tek's immunodot strips, from the deposit of the sample to the final colour development. The maximum capacity is of 24 strips which are incubated simultaneously. Each strip is associated with a cartridge containing the various reagents making it possible to carry out the test. The BlueDiver Instrument has a barcode reader which controls the correct association between a strip and its cartridge. Prior training is strongly recommended (see your distributor). Please consult The User Manual before using the BDI I.

**The BlueScan and Dr Dot Software:**

The BlueScan and Dr Dot Software are intended for reading test results of D-tek immunodot strips. The Dr Dot Software and BlueScan scanner have to be used in combination.

The scanner has been specifically developed for the reading of the strips with "BlueDiver" design. Based on the image of the scanned strips, the Dr Dot Software converts the intensity of each dot/line into a numerical value (the numerical scale is based on a greyscale). Results are expressed in arbitrary units (from 0-100). 1-24 strips can be read. Prior training is strongly recommended (see your distributor).

Please contact your distributor to obtain the latest version of Dr Dot Software.

Please consult The User Manual before using the BlueScan and Dr Dot Software.

**BDI II:**

The BlueDiver Instrument II is an instrument that performs the various steps of samples pipetting, incubation, washing, drying and reading of D-tek's immunodot strips, from the deposit of the sample tube to the final reading of the strips.

The maximum capacity of the BDI II is of 24 strips which are incubated simultaneously. Each strip is associated with a cartridge containing the various reagents making it possible to carry out the test. The BDI II has a barcode reader which controls the correct association between a strip and its cartridge.

It includes the BlueScan and Dr Dot reading system.

Prior training is mandatory (see your distributor).

Please consult The User Manual before using the BDI II

**Other Material:** Micropipettes, absorbent paper, protective equipment (see point 6).

**5. STORAGE**

The test kit must be stored at a temperature between +2°C to +8°C throughout its validity period (see expiration date on the kit). Do not freeze.

After initial opening of the kit, unused reagent cartridges must be stored at 2-8°C protected from (sun)light preferably inside the original kit box.

Unused strips have to be placed back into the provided pouches, sealed and stored at 2-8°C preferably inside the original kit box. When stored properly, all test kit components are stable until the indicated expiry date.

**6. SAFETY PRECAUTIONS**

1. All reagents are for in vitro diagnostic and professional use only. The test kit should be processed by trained technical staff only.

2. The reagents in the kit are considered as not dangerous, as the concentrations of potentially dangerous chemicals are below the thresholds specified by European regulations. More information is available on the MSDS of the kit (available upon request or on the D-tek website [www.d-tek.be](http://www.d-tek.be)).

Nevertheless, the product contains preservatives which may have (in their given concentration), slightly polluting properties or causing skin sensitization. Therefore, contact with the skin, eyes or mucous membranes should be avoided. As with any chemical containing specific hazards, the product/components of the product should only be handled by qualified personnel and with the necessary precautions.

3. Patient samples should be handled as if they were capable of transmitting infectious diseases; they therefore require suitable protection (gloves, laboratory coat, goggles). In any case, GLP should be applied with all the general or individual safety rules in force.

4. Waste disposal: Patient samples, incubated test strips and used cartridges should be handled as infectious waste. The boxes and other containers do not need to be collected separately, unless stated otherwise in official regulations.

**7. RECOMMENDATIONS**

1. D-tek and its authorized distributors cannot be held responsible for damages caused indirectly or due to: a change or modification in the indicated procedure, an improper use of the kit and / or the use of an incomplete or damaged kit. The use of this kit is reserved for qualified technical personnel only.

2. D-tek's responsibility is limited in all cases to the replacement of the kit.

3. In the event of a serious incident (injury, deterioration in health, or death) with this IVD device, please report it immediately to the manufacturer (see address below) and to the competent authority in your country.

**8. SAMPLE COLLECTION, HANDLING AND STORAGE**

The test should be used on recently collected sera samples only! Sera with particles should be centrifuged at low speed. Blood samples should be collected in dry tubes. Please avoid using a pool of different sera, as this can lead to inconsistent results (see point 10.4). After separation, the serum samples should be used immediately or aliquoted and stored at 2-8 ° C (for storage for a few days) or frozen at -20°C (for longer storage periods). Repeated freezing/ thawing cycles of the samples must be avoided.


**9. ASSAY PROCEDURE****BASIC INFORMATION, HANDLING AND TIPS:****TEST PROCESS principle:**

After the manual loading of the strips and reagent cartridges, the incubation and washing steps are automatically processed by the BDI; the continuous up and down agitation of the strips in the wells of the ready-to-use reagent cartridges ensures an efficient circulation of the fluids over the strips. The whole test procedure is run at room temperature.

**Description of the STRIPS:**

**The reactive (front) side** of the strips is coated with antigens which appear as faint blue dots. This coloration ensures that all antigens have been correctly spotted onto the membrane. The coloration disappears during the processing of the test. This front side also displays a strip number and a 2-dimensional square barcode for traceability of the strips after removal from the BDI at the end of the test.

**The non-reactive (back) side** of the strips displays both alphanumeric and bar-coded information for identification of the strip type and lot number by the BDI.

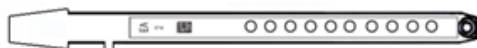

The strips must be manually inserted into the dedicated clamp before starting the automated process (see 9.1 and 9.2 *Test Preparation* hereafter). During this operation, avoid touching the membrane zone of the strips with fingers. Always wear laboratory gloves and use the plastic parts (strip support) for handling or manipulation.

**Description of the CARTRIDGES: (see image page 1)**

The reagent cartridges are composed of 7 different wells filled with ready-to-use reagents. The cartridges are sealed, and the reagent wells are hermetically separated. The sealing has to be removed before starting the test. Once opened, manipulate the cartridges with care in order to avoid reagent spilling and contamination from well to well.

The rear (back) side of the cartridges is labelled with both alphanumeric and bar-coded information for identification of the cartridge type and lot number by the BDI.

The cartridges must be manually loaded onto the dedicated cartridge holder before starting the automated process ((see 9.1 and 9.2 *Test Preparation* hereafter). The front and rear (back) sides of the cartridges have, respectively, a bottom triangular and two (bottom + top) square plastic edges for secure position and orientation into the holder.

**Description of the CONTROLS:**

The **Positive Control or RC (Reaction Control)** consists of a protein fixing all the immunoglobulins present in the test sample. If the test has been carried out correctly, this control will show a colouring at the end of the test (with an intensity depending on the effective concentration of immunoglobulins in the sample).

The absence of any colouring of this dot at the end of the test may indicate that the sample has not been pipetted on the strip (see 10.4 *Troubleshooting*).

The **Negative Control or CO (Cut-Off Control)** consists of a protein reacting with the enzymatic substrate and with certain constituent elements of the tested sample. If the test has been carried out correctly, this control is coloured at the end of the test, with a signal depending on the kinetics of the substrate and the characteristics of the sample. The intensity of this control serves as a threshold value for the final interpretation of the results (see 10 *INTERPRETATION OF RESULTS*).

**STRIPS/CARTRIDGES association**

The strips and cartridges of a same test kit share the same lot number and are dedicated to be associated in lot-specific pairs. Do not associate a strip and a cartridge with different lot numbers as this will be detected as an invalid setting by the BDI and will stop the process.

As long as each strip/cartridge pair is valid, the BDI can process strips/cartridges associations of different kits; However, only kits having the same protocol number (same incubation time and sequence) can be processed together in one same run (please refer to the protocol number indicated under the kit reference at the top of first page).

**9.1 Test preparation on BDI I**

Before any usage of the BDI, please refer to the manual of use supplied with the instrument.

- Allow all kit components to equilibrate at room temperature (+18°C to +25°C) before use.
- A working list (either edited from Dr Dot Software, or external) should always be prepared for easy loading and correct association of strips, cartridges and patient samples.
- Make sure that the cartridge holder is fixed in its emplacement in the BDI.
- Make sure that the BDI is plugged in.

The following steps sequence summarizes the loading and preparation of the BDI, test strips, reagent cartridges and patient samples before starting the test. For detailed information or in case of any problem met at one of the following steps, please refer to the Manual of Use of the BDI.

1. Switch ON the BDI and wait a few seconds until the date and time are displayed on the touch screen.
2. Confirm the correct Date and Time by pressing **V** on the touch screen (in case of first use or for reset, refer to the manual of use of the BDI) → "**Initialize?**" is displayed on the screen.
3. Confirm Initialization by pressing **V** on the touch screen → the horizontal arm of the instrument automatically moves forward to a central (stand-by) position → "**Load strips (24)**" is displayed on the screen.
4. (**Do not set nor confirm the number of strips at this step**). Remove the clamp from its emplacement on the arm by gently pulling it upwards and load the strips to be tested: handle the clamp with numbered side facing up (open position) and insert the strips, also with numbered (reactive) side facing up, by slipping the upper plastic part (tongue) into the dedicated holes of the clamp. Apply a gentle pressure to ensure that the plastic tongue has reached the bottom end of the hole.  
Notes:  
Always start loading into position 1 of the clamp (left side) and do not leave empty spaces between the strips!  
After complete loading, check visually the vertical, horizontal and lateral alignment of the strips. Any obvious misalignment should be corrected by unloading the strip(s) from the clamp and loading them again.  
Be careful: any plastic bits remaining after breaking apart the individual strip holders may hinder the processing on the instrument and/or the reading with the BlueScan scanner; please remove them with scissors.
5. Replace the clamp in its emplacement on the arm by gently pushing it downwards.
6. Set the number of loaded strips using the up and down arrows on the touch screen.
7. Confirm the number of loaded strips by pressing **V** on the touch screen → the horizontal arm automatically moves backward to stand over the alignment holes of the cartridge holder → "**Check alignment**" is displayed on the screen.
8. Use the "JOG" function on the screen to check the correct alignment of the strips: maintain a gentle pressure on the down arrow on the touch screen until the bottom of the strips enters into the alignment holes of the cartridge holder. If correctly aligned, the strips will not touch the outlines of the holes.  
Note:  
In case of misalignment (contact of the strips with the cartridge holder), please refer to the Manual of Use of the BDI).
9. Confirm the correct alignment of the strips by pressing **V** on the touch screen → the BDI lowers the strips completely into the alignment holes and reads the barcodes of the strips → after complete barcode reading, "**Load reagent**" is displayed on the touch screen.  
Note:  
In case of failure to read one or more strip barcode(s) (flashing LED at the unread position), please refer to the Manual of Use of the BDI.
10. Unseal the reagent cartridges and insert them under their respective strips in the dedicated notches of the cartridge holder.
11. Confirm complete loading by pressing **V** on the touch screen → the BDI reads the barcodes of the cartridges and checks the correct association with the strips → after complete barcode reading, the number of strips (validated strips/cartridges associations) is displayed on the screen.  
Note:

- In case of failure to read one or more cartridge barcode(s), or in case of detection of a wrong strip/cartridge association (flashing LED at the corresponding position), please refer to the Manual of Use of the BDI.
12. Confirm the number of strips by pressing **V** on the touch screen → the protocol number identified on the barcodes is displayed on the screen (**Protocol ID xx**).
  13. Confirm the protocol number by pressing **V** on the touch screen → **"Please close cover."** is displayed on the screen.
  14. Close the cover of the BDI and confirm closing by pressing **V** on the touch screen → the BDI proceeds to a first washing (pre-treatment) step by incubating the strips into the 2<sup>nd</sup> well of the cartridges (processing time: 1 minute) → At the end of the wetting step, **"Please open cover."** is displayed on the screen.
  15. Open the cover of the BDI and confirm opening by pressing **V** on the touch screen → the horizontal arm automatically moves forward to the front of the instrument and swings the strips to an oblique position → **"Dry strips"** is displayed on the screen.
  16. Dry the strips by gently applying absorbent paper onto the basis of the bottom small cavity (sample loading hole).
  17. Confirm drying by pressing **V** on the touch screen → **"Apply samples"** is displayed on the screen.
  18. Apply samples by pipetting 10 µl of patient serum into the bottom sample loading holes of the strips.  
Note:  
 If preferred the 10µl of the serum can be directly pipetted into the Diluent Buffer ("Well I") of the cartridge. This operation can be done at any time from opening of the cartridges (see 9.1.10).
  19. Confirm samples' loading by pressing **V** on the touch screen → **"Please close cover"** is displayed on the screen.  
 Close the cover of the BDI and confirm closing by pressing **V** on the touch screen → the BDI starts the test automatically by proceeding the steps sequence of the protocol (see 9.3). After completion of the process, the clamp moves to a central (stand-by) position in the BDI to allow easy manipulation of the clamp. The instrument beeps and **"Finished test"** is displayed on the screen.
  20. Gently apply absorbent paper onto the basis of the strips to remove liquid from the bottom small cavity (sample loading hole) and allow the strips to dry for 30 minutes before interpretation of the results. The interpretation has to be done in the 24 hours following the test processing. In case of use of the BlueScan for help of results interpretation, please leave the processed strips attached to the clamp

#### TEST DATA REGISTRATION

The test protocol can be downloaded by pressing the USB stick symbol and following the indications on the screen (Insert USB → Writing USB → Remove USB). This step is not obligatory but is highly recommended for traceability and regulatory matters.

#### 9.2 Test preparation on BDI II

Before any usage of the BDI II, please refer to the manual of use supplied with the instrument.

- Allow all kit components to equilibrate at room temperature (+18°C to +25°C) before use.
- All the preparatory steps requiring the operator's intervention are clearly indicated in the BDI's user interface. It is the instrument which indicates the number and type of tests to be run according to the indications made by the operator at the sample identification stage.  
 The operator is guided by the user interface from the insertion of the samples and kits to be tested onwards, to the final interpretation of the results.
- Remember to open the reagent cartridges before inserting them into the holder.

#### 9.3 Test Processing (Protocol 02 for all D-tek immunodot kits on BDI I and BDI II):

Step	Description	Processing time
01.	The strips are incubated into the 1 <sup>st</sup> well of the cartridge ( <i>Diluent Buffer</i> ). Upon contact with the liquid in the wells and agitation, the pre-loaded patients' samples (see 9.1.18) are released from the small cavity at the bottom of the strips and are diluted in the buffer.	30 min
02.	The clamp moves forwards and the strips are incubated into the 2 <sup>nd</sup> well of the cartridge ( <i>Wash Buffer</i> )	2 min
03.	The clamp moves forwards and the strips are incubated into the 3 <sup>rd</sup> well of the cartridge ( <i>Wash Buffer</i> )	2 min
04.	The clamp moves forwards and the strips are incubated into the 6 <sup>th</sup> well of the cartridge ( <i>Wash Buffer</i> )	2 min
05.	The clamp moves backwards and the strips are incubated into the 5 <sup>th</sup> well of the cartridge ( <i>Conjugate</i> )	10 min
06.	The clamp moves backwards and the strips are incubated into the 4 <sup>th</sup> well of the cartridge ( <i>Wash Buffer</i> )	2 min
07.	The clamp moves backwards and the strips are incubated into the 3 <sup>rd</sup> well of the cartridge ( <i>Wash Buffer</i> )	2 min
08.	The clamp moves backwards and the strips are incubated into the 2 <sup>nd</sup> well of the cartridge ( <i>Wash Buffer</i> )	2 min
09.	The clamp moves forwards and the strips are incubated into the 7 <sup>th</sup> well of the cartridge ( <i>Substrate</i> )	10 min
10.	The clamp moves backwards and the strips are incubated into the 6 <sup>th</sup> well of the cartridge ( <i>Wash Buffer</i> )	2 min

#### 10. INTERPRETATION OF RESULTS

For *BDI I*: a visual (qualitative) interpretation of the results is possible, however the use of the BlueScan scanner and the Dr Dot software is generally recommended for more precision and for a semi-quantitative interpretation.

For *BDI II*: the semi-quantitative interpretation of the results is systematically performed at the end of the test.

**IMPORTANT NOTICE: The positivity of all parameters of this kit is NOT possible and in such a case the test is not valid. An additional test has to be performed to establish the diagnosis**

##### 10.1. Qualitative Interpretation

1. Remove the clamp from the BDI and unload the strips from the clamp.
2. Place the strips with reactive face up onto the marked fields of the Diver Interpretation Template provided with the kit. This will indicate the respective positions of the different controls and antigens on the membrane.
3. The first upper dot (Positive Control) must be positive for all patients.  
*Only a clearly coloured Positive Control Dot ensures your results are valid and operation was correct and/or kit components were not degraded. If the first upper dot is not coloured, the test has failed and cannot be interpreted further.*
4. Compare the **specific antigen dots** to the **Negative Control Dot** (which always is the last bottom dot). The colour intensity of the Antigen dots is directly proportional to the titer of the specific antibody in the patient sample.  
*The colour intensity of the Negative Control Dot may vary depending on the sample characteristics. If the sample is free of interfering substances the Negative Control Dot may be even close to uncoloured. In contrast, a highly coloured Negative Control Dot indicates a high rate of unspecific binding in the sample.*

**POSITIVE RESULT:**

A sample is positive for a specific antibody if the colour intensity of the corresponding Antigen dot is higher than the intensity of the Negative Control Dot.

**NEGATIVE RESULT:**

A sample is negative for a specific antibody if the colour intensity of corresponding Antigen dot is lower than or equal to the intensity of the Negative Control Dot.

NB: A weak coloration of an antigen dot, when close to the colour intensity of the Negative Control Dot may be difficult to differentiate by visual inspection only. In such cases, it is recommended to use Dr Dot Software and scanning system (see 10.2) and refer to the corresponding instructions for more accurate interpretation.

**10.2 Results semi-quantification: use of Dr Dot Software and Scanning system**

The BlueScan scanner is an especially designed system for the reading of D-tec immunodot strips. It allows precise and easy insertion of test strips.

The Dr Dot Software allows a semi-quantification of results. Based on the image obtained, each result will be quantified in grayscale value and compared to the reference scale integrated in the BlueScan Cover.

These grayscale intensities will be transformed and displayed in arbitrary units (AU, from 0 to 100) based on the intensities of the controls (RC and CO, see point 9) present on the strip, according to the following conversion formula:

$$\text{Result of antigen X (AU)} = \frac{\text{Grayscale intensity of antigen X} - \text{Grayscale intensity of CO}}{\text{Grayscale intensity of RC} - \text{Grayscale intensity of CO}} * 100$$

1. Remove the clamp from the BDI. Leave the processed strips attached to the clamp. Be careful: the strips have to be completely dry before starting the scanning step!
2. Insert the clamp, the reactive side of the strips facing down, into the dedicated emplacement in the cover of the BlueScan scanner.
3. Start scanning the strips using the Dr Dot Software.
4. The software semi-quantifies the results, and the interpretation of the obtained values is as follows

Dr Dot arbitrary unit (AU)	Interpretation
< 5	Negative
5 – 10	Equivocal
>10	Positive

For detailed information about the BlueScan and Dr Dot Software please refer to the Manual of Use of your Dr Dot Software

**10.3 Important recommendations for the interpretation of results**

1. D-tec's kits constitute a diagnostic aid. In consequence, no diagnosis can be established solely on the basis of our kits. The results should always be interpreted by taking into account the clinical examination, the patient's history and the results obtained by other methods.  
No single technique can rule out the possibility of false positive or false negative results. With this in mind, an indirect immunofluorescence test should, as far as possible, be carried out prior to the use of a BlueDiver Dot kit (immunofluorescence being recognized as a reference method in autoimmunity).
2. The intensity of a result is not necessarily related to the degree of intensity of the disease, but rather to the level of antibodies detected.
3. Low titers of auto-antibodies may occur in healthy patients. For this reason, low positive results (close to the CO, between 5 and 10 DrDot AU), although valid, should be considered equivocal. In such cases, the retesting of the patient, preferably by using a new sample, is recommended. If the result remains equivocal on retesting, other diagnostic tests and/or clinical information should be used to help determine the autoimmune status of the patient.
4. For various reasons, and under certain conditions, the kit may show a defect in performance (see 10.4 *Troubleshooting*). In such cases, the results are not valid and cannot be interpreted. It is recommended to repeat the test. If the error persists, please contact your distributor.
5. The intensity of the results may decrease when the device is used at the end of its life. However, the performance of the kit is not affected (detection of positives and negatives) under normal conditions of use and storage.
6. Sequential sampling (at different dates) of an autoimmune patient can sometimes lead to different results from one sample to another. This difference can have several reasons: the patient's treatment, the evolution of the disease, or a seroconversion. In the specific case of seroconversion, the result can be positive for an auto-antibody in an early sampling of the patient, and become positive for another auto-antibody in a later sampling of the same patient.

**10.4 Troubleshooting**

Problem	Possible causes + Action
Discrepancy of results as compared to a reference method	<ul style="list-style-type: none"> <li>- Use                             <ul style="list-style-type: none"> <li>- incorrect pipetting of serum</li> <li>- incorrect volume dispensed</li> <li>- Use of two different samples of the same patient (see point 10.3.6) or wrong sample handling/storage between tests</li> <li>- erroneous visual interpretation</li> <li>- erroneous DrDot reading</li> <li>➔ repeat the test</li> </ul> </li> <li>- Material                             <ul style="list-style-type: none"> <li>- Interfering substance in the sample</li> <li>- Sample is a pool of different human sera</li> <li>➔ repeat the test and confirm by other methods</li> </ul> </li> <li>- Method                             <ul style="list-style-type: none"> <li>- intrinsic performance of the kit (see 11.2 <i>Analytical sensitivity and specificity</i>)</li> <li>- expired kit</li> </ul> </li> </ul>

	<ul style="list-style-type: none"> <li>- stability problem</li> </ul> <p><b>Please contact your distributor for any further technical support requests.</b></p>
Different results in the same batch or between several batches	<ul style="list-style-type: none"> <li>- Use <ul style="list-style-type: none"> <li>- incorrect pipetting of serum</li> <li>- incorrect volume dispensed</li> <li>- erroneous visual interpretation or</li> <li>- bad DrDot reading</li> <li>→ repeat the test</li> </ul> </li> <li>- Method <ul style="list-style-type: none"> <li>- intrinsic performance of the kit (see 11.1 <i>Repeatability and Reproducibility</i>)</li> </ul> </li> </ul>
Contamination between neighbouring strips	<ul style="list-style-type: none"> <li>- Use <ul style="list-style-type: none"> <li>- incorrect pipetting of serum</li> <li>→ repeat the test</li> <li>- incorrect verticality of the strips on the BDI</li> <li>→ correct the verticality</li> </ul> </li> </ul> <p><b>Please contact your distributor for any further technical support requests.</b></p>
RC absent or weak	<ul style="list-style-type: none"> <li>- Use <ul style="list-style-type: none"> <li>- Serum not pipetted at all</li> <li>→ repeat the test</li> <li>- Patient with immunoglobulin deficiency</li> <li>→ repeat the test to confirm patient status</li> <li>- Damaged reagents</li> <li>→ check the integrity of the reagents</li> <li>→ contact your supplier if you suspect a problem</li> <li>- Spot not on the strip</li> <li>→ count the number of dots on the strip; if not correct, contact your supplier</li> </ul> </li> </ul>
CO absent	<ul style="list-style-type: none"> <li>- damaged reagents <ul style="list-style-type: none"> <li>→ check the integrity of the reagents, contact your dealer if you suspect a problem</li> </ul> </li> <li>- Spot absent from the strip <ul style="list-style-type: none"> <li>→ count the number of spots present on the strip, contact your distributor in case of incorrect number</li> </ul> </li> </ul>
Non-specific bindings / high background / high CO value	<p>Suspected presence of a contaminant or an interfering substance in the patient sample</p> <p>→ repeat the test and confirm through another method</p> <p><b>Please contact your distributor for any further technical support requests.</b></p>
Barcode of the strips or cartridges cannot be read	Manufacturing problem, please contact your distributor
Kit content incorrect	Manufacturing problem, please contact your distributor
Positive results for all the biomarkers of the kit	Problem with reagents, please contact your distributor

**NOTE:**

The major residual risks of the kit, as given in the risk analysis of the kit at the end of design (after mitigation), are the following:

- 1) Risk of false results based on a pipetting error (bad serum)
- 2) Risk of false results based on an interfering substance contained in the sample

**11. PERFORMANCES**

**11.1 Repeatability and Reproducibility**

Reference samples were tested for each antibody in successive statistically representative series, both in the same test as in different tests and between different batches in order to calculate the intra-assay, inter-assay and inter-lot variations respectively. In all the cases, the variations in colour intensity, when semi-quantified on the Dr Dot software, were within the following expected limits:

- CV ≤ 10% for intra-assay runs
- CV ≤ 15% for inter-assay runs
- CV ≤ 20% for inter-lot runs

**11.2 Analytical sensitivity**

Measurement range (semi-quantified results): From 0 AU (negative) to 100 AU (high positive).

Limit of detection: the lowest measured value of the test is 5 AU (considered as equivocal following the interpretation algorithm, see point 10.2)

As not any international standard is available for the auto-antibodies, trueness of measurement and linearity are not applicable on this product.

**11.3 Analytical specificity**

1. The main known interfering substances were tested on each biomarker of the present kit. For each concentration of interfering substance tested, the difference between the result of the sample without the interfering substance and the result obtained in the presence of the interfering substance did not exceed 15%.

Interfering substance	Maximum Concentration	Intermediate Concentration	Minimum Concentration	Difference <15%
Bilirubin	100 mg/dL	50 mg/dL	25 mg/dL	Yes
Haemoglobin	200 mg/dL	100 mg/dL	50 mg/dL	Yes
Cholesterol	224.3 mg/dL	112 mg/dL	56 mg/dL	Yes
Rheumatoid factor IgM	~500IU/ml	~300IU/ml	~100IU/ml	Yes

Note: It is impossible to test all the possible interfering substances described in the literature. Other interferences, amongst others drug-induced interferences, are possible.

2. The high analytical specificity of the test is guaranteed by the quality of the antigen used. This kit detects IgG antibodies against M2/nPDC, gp210, sp100, LKM1, LC1, SLA and F-actin. No cross reactions with other autoantibodies have been found.

#### 11.4 Clinical sensitivity and specificity

Characterized samples (confirmed positive or negative for specific antibodies by reference laboratories and/or methodologies) were assayed following the test instructions. Sensitivity and Specificity were calculated from the results generated by the Dr Dot Software. A detailed clinical report is available upon request.

<b>M2/nPDC</b>	<b>gp210</b>	<b>sp100</b>	<b>LKM1</b>																								
<table border="1"> <thead> <tr> <th>+</th> <th>-</th> </tr> </thead> <tbody> <tr> <td>True positive 42</td> <td>False positive 0</td> </tr> <tr> <td>False negative 0</td> <td>True negative 109</td> </tr> </tbody> </table> <p><b>Sensitivity</b> <math>\frac{42}{42} = &gt; 99\%</math></p> <p><b>Specificity</b> <math>\frac{109}{109} = &gt; 99\%</math></p>	+	-	True positive 42	False positive 0	False negative 0	True negative 109	<table border="1"> <thead> <tr> <th>+</th> <th>-</th> </tr> </thead> <tbody> <tr> <td>True positive 3</td> <td>False positive 0</td> </tr> <tr> <td>False negative 0</td> <td>True negative 53</td> </tr> </tbody> </table> <p><b>Sensitivity</b> <math>\frac{3}{3} = &gt; 99\%</math></p> <p><b>Specificity</b> <math>\frac{53}{53} = &gt; 99\%</math></p>	+	-	True positive 3	False positive 0	False negative 0	True negative 53	<table border="1"> <thead> <tr> <th>+</th> <th>-</th> </tr> </thead> <tbody> <tr> <td>True positive 10</td> <td>False positive 0</td> </tr> <tr> <td>False negative 1</td> <td>True negative 45</td> </tr> </tbody> </table> <p><b>Sensitivity</b> <math>\frac{10}{11} = 91\%</math></p> <p><b>Specificity</b> <math>\frac{45}{45} = &gt; 99\%</math></p>	+	-	True positive 10	False positive 0	False negative 1	True negative 45	<table border="1"> <thead> <tr> <th>+</th> <th>-</th> </tr> </thead> <tbody> <tr> <td>True positive 12</td> <td>False positive 0</td> </tr> <tr> <td>False negative 0</td> <td>True negative 119</td> </tr> </tbody> </table> <p><b>Sensitivity</b> <math>\frac{12}{12} = &gt; 99\%</math></p> <p><b>Specificity</b> <math>\frac{119}{119} = &gt; 99\%</math></p>	+	-	True positive 12	False positive 0	False negative 0	True negative 119
+	-																										
True positive 42	False positive 0																										
False negative 0	True negative 109																										
+	-																										
True positive 3	False positive 0																										
False negative 0	True negative 53																										
+	-																										
True positive 10	False positive 0																										
False negative 1	True negative 45																										
+	-																										
True positive 12	False positive 0																										
False negative 0	True negative 119																										
<table border="1"> <thead> <tr> <th>+</th> <th>-</th> </tr> </thead> <tbody> <tr> <td>True positive 3</td> <td>False positive 0</td> </tr> <tr> <td>False negative 0</td> <td>True negative 29</td> </tr> </tbody> </table> <p><b>Sensitivity</b> <math>\frac{3}{3} = &gt; 99\%</math></p> <p><b>Specificity</b> <math>\frac{29}{29} = &gt; 99\%</math></p>	+	-	True positive 3	False positive 0	False negative 0	True negative 29	<table border="1"> <thead> <tr> <th>+</th> <th>-</th> </tr> </thead> <tbody> <tr> <td>True positive 3</td> <td>False positive 0</td> </tr> <tr> <td>False negative 0</td> <td>True negative 30</td> </tr> </tbody> </table> <p><b>Sensitivity</b> <math>\frac{3}{3} = &gt; 99\%</math></p> <p><b>Specificity</b> <math>\frac{30}{30} = &gt; 99\%</math></p>	+	-	True positive 3	False positive 0	False negative 0	True negative 30	<table border="1"> <thead> <tr> <th>+</th> <th>-</th> </tr> </thead> <tbody> <tr> <td>True positive 12</td> <td>False positive 0</td> </tr> <tr> <td>False negative 1</td> <td>True negative 137</td> </tr> </tbody> </table> <p><b>Sensitivity</b> <math>\frac{12}{13} = 92\%</math></p> <p><b>Specificity</b> <math>\frac{137}{137} = &gt; 99\%</math></p>	+	-	True positive 12	False positive 0	False negative 1	True negative 137	<p>Note: Sensitivity and specificity values of 100 % are strictly related to sample cohorts used in clinical evaluations. In theory, a diagnostic kit shouldn't be considered to be 100% sensitive or specific (at least &gt; 99%).</p>						
+	-																										
True positive 3	False positive 0																										
False negative 0	True negative 29																										
+	-																										
True positive 3	False positive 0																										
False negative 0	True negative 30																										
+	-																										
True positive 12	False positive 0																										
False negative 1	True negative 137																										

#### 11.5 Auto-antibodies diagnostic values

Anti-M2/nPDC	<p>Anti-M2/nPDC are marker antibodies of primary biliary cholangitis (PBC) and are detectable in nearly 95% of cases. They count towards the three diagnostic criteria for PBC.</p> <p>Although they are highly specific for PBC, Anti-M2/nPDC can also be detected in patients with chronic inflammatory rheumatic diseases. It is believed that these patients are at an increased risk of developing PBC in addition to the underlying disease. Particularly in Anti-M2/nPDC positive CREST variant of systemic sclerosis there is an increased risk of PBC development (Fregeau et al., 1988; Zurgil et al., 1992). In patients with SLE, the presence of Anti-M2/nPDC is significantly associated with increased aminotransferase (Li et al., 2006).</p> <p>Anti-M2/nPDC are detectable in 3–6% of autoimmune hepatitis (AIH) type 1 patients. These are most often cases of an AIH/PBC overlap syndrome. AIH/PBC overlap should be considered when the ALP to aminotransferase ratio is less than 1.5, IgG is elevated and the SMA are present with a titer greater than 1:80 (Bowlus &amp; Gershwin, 2014).</p> <p>Anti-M2/nPDC can be predictive. They can appear years before manifestations of PBC. Individuals with persistently high Anti-M2/nPDC antibody levels have a higher risk of developing PBC. Prospective studies have shown that 76% of asymptomatic Anti-M2/nPDC positive patients over a period of observation from 11–24 years are diagnosed with PBC (Metcalf et al., 1996). The prevalence of Anti-M2/nPDC in the first-degree relatives of PBC patients is high (13.1%) (Nakamura et al., 2014).</p> <p>Anti-M2/nPDC titers do not change over time and are not associated with disease severity or progression (Benson et al., 2004). On the other hand some groups have been shown that the Anti-M2/nPDC titer decrease with the treatment with UDCA (Nakamura et al., 2014).</p> <p>Anti-M2/nPDC persist following liver transplantation.</p>
Anti-gp210	<p>Anti-gp210 antibodies are highly specific for primary biliary cholangitis (PBC) and are detectable by enzyme immunoassay in 10-45% of PBC patients with a specificity of 99.5%.</p> <p>They are rarely or very rarely observed in autoimmune hepatitis, chronic hepatitis B (12.6%), rheumatoid arthritis, polymyositis or Sjögren's syndrome. Their possible predictive value is currently unknown.</p> <p>The titer of anti-gp210 antibodies depends on disease activity or stage progression.</p> <p>Anti-gp210 antibodies are associated with extrahepatic manifestations, as arthritis. They are also considered as prognostic markers of a poor outcome and correlated to a higher risk of hepatic failure.</p> <p>Anti-gp210 antibodies persist after liver transplantation, and are therefore an unsuitable marker of possible disease recurrence.</p>
Anti-sp100	<p>Anti-sp100 antibodies are specific (97%) for primary biliary cholangitis (PBC) with a diagnostic sensitivity of 20-40%. These autoantibodies are found relatively often (48%) in the group of AMA negative patients with a clinically and histologically proven PBC.</p> <p>Anti-sp100 antibodies seem to be associated with urinary tract infections. 74% of PBC patients with urinary tract infections are positive for anti-sp100 antibodies (Bogdanos et al., 2003).</p> <p>In low frequencies, anti-sp100 antibodies have been found in rheumatoid disease (3% in rheumatoid arthritis, up to 10% in systemic lupus erythematosus, in ~5% in systemic sclerosis, in 2% in Sjögren's syndrome).</p> <p>Anti-sp100 antibodies persist following liver transplantation, and are therefore an inappropriate marker for possible disease recurrence.</p>
Anti-LKM1	<p>LKM1 antibodies are marker antibodies of autoimmune hepatitis (AIH) type 2 and are included in the diagnostic AIH criteria of the International Autoimmune Hepatitis Group with a sensitivity of 90-95% in (mainly) young patients. They also are part of the simplified criteria of AIH. Patients with AIH type 2 are typically ANA and SMA negative. In primary biliary cholangitis (PBC) and primary sclerosing cholangitis</p>



	(PSC), LKM1 antibodies are rarely detected. LKM1 antibodies occur in ~50–60% of cases together with LC1 antibodies, however they can also be detected in isolation.
Anti-LC1	LC1 antibodies are detectable in 30–59% of patients with autoimmune hepatitis (AIH) type 2 and are a diagnostic criterion of the International Autoimmune Hepatitis Group. They are predominantly found in children and younger patients and are often associated with LKM1 antibodies. In 50–60% of LKM1 antibody positive patients, LC1 antibodies are also detected as a second marker antibody of AIH type 2. In ~10% of AIH type 2 patients however, LC1 antibodies are the only marker antibodies found. In pediatric AIH type 2, LC1 antibodies are more frequent (59%) than in adults (28.6%).
Anti-SLA	SLA/LP antibodies are highly specific for autoimmune hepatitis (AIH) type 3. Although the definition of AIH type 3 is controversial, as it is clinically and therapeutically not different from AIH type 1, it is clearly a separate entity due to the SLA/LP antibodies. The diagnostic sensitivity has been reported as 19–33%. Their positive predictive value is nearly 100%.
Anti-F-actin	High titers of anti-F-actin are marker antibodies and are accordingly diagnostic criteria of the International Autoimmune Hepatitis Group (three points in the scoring system for a titer >1: 80, two points for 1:80 and one point for 1:40) for autoimmune hepatitis (AIH) type 1. They are also part of the simplified criteria of AIH. They are very often associated with anti-nuclear antibodies (ANA), however they can be isolated positive in ~35% of AIH type 1 patients. The diagnostic sensitivity and specificity for AIH type 1 are ~80% and 96%, respectively. Therefore, a negative anti-F-actin result cannot completely rule out AIH. The titer has a limited correlation with disease activity. Only high titers >1:80 are associated with the disease activity. Neither the antibody titer at diagnosis nor the antibody behavior in the course of the disease are prognostic markers. Note: In children a titer of 1:20 can be diagnostically relevant. Most low titers of anti-F-actin can be found in viral infections, such as infectious mononucleosis, chronic hepatitis C (8–10%), however also in rheumatic diseases, primary biliary cholangitis (PBC) (22%), patients with alcoholic liver disease (3–16%) and neoplastic disease. Their prevalence in healthy individuals is ~5%.

**Publication references :**

- 1: Chen BH, Wang QQ, Zhang W, Zhao LY, Wang GQ. Screening of anti-mitochondrial antibody subtype M2 in residents at least 18 years of age in an urban district of Shanghai, China. *Eur Rev Med Pharmacol Sci.* 2016 May;20(10):2052-60. PMID: 27249604.
- 2: Pang SY, Dai YM, Zhang RZ, Chen YH, Peng XF, Fu J, Chen ZR, Liu YF, Yang LY, Wen Z, Yu JK, Liu HY. Autoimmune liver disease-related autoantibodies in patients with biliary atresia. *World J Gastroenterol.* 2018 Jan 21;24(3):387-396. doi: 10.3748/wjg.v24.i3.387. PMID: 29391761; PMCID: PMC5776400.
- 3: Zandanell S, Strasser M, Feldman A, Tevini J, Strebingner G, Niederseer D, Pohla-Gubo G, Huber-Schönauer U, Ruhaltinger S, Paulweber B, Datz C, Felder TK, Aigner E. Low rate of new-onset primary biliary cholangitis in a cohort of anti-mitochondrial antibody-positive subjects over six years of follow-up. *J Intern Med.* 2020 Apr;287(4):395-404. doi: 10.1111/joim.13005. Epub 2019 Dec 4. PMID:31802567; PMCID: PMC7154539.
- 4: Calise SJ, Zheng B, Hasegawa T, Satoh M, Isailovic N, Ceribelli A, Andrade LEC, Boylan K, Cavazzana I, Fritzler MJ, de la Torre IG, Hiepe F, Kohl K, Selmi C, Shoenfeld Y, Tincani A, Chan EKL; IUIS Autoantibody Standardization Committee. Reference standards for the detection of anti-mitochondrial and anti-rods/rings autoantibodies. *Clin Chem Lab Med.* 2018 Sep 25;56(10):1789-1798. doi: 10.1515/cclm-2017-1152. PMID: 29478040; PMCID: PMC8128709.
- 5: Amin K, Rasool AH, Hattem A, Al-Karboly TA, Taher TE, Bystrom J. Autoantibody profiles in autoimmune hepatitis and chronic hepatitis C identifies similarities in patients with severe disease. *World J Gastroenterol.* 2017 Feb 28;23(8):1345-1352. doi: 10.3748/wjg.v23.i8.1345. PMID: 28293081; PMCID: PMC5330819.
- 6: Deng CW, Wang L, Fei YY, Hu CJ, Yang YJ, Peng LY, Zeng XF, Zhang FC, Li YZ. Exploring pathogenesis of primary biliary cholangitis by proteomics: A pilot study. *World J Gastroenterol.* 2017 Dec 28;23(48):8489-8499. doi: 10.3748/wjg.v23.i48.8489. PMID: 29358857; PMCID: PMC5752709.
- 7: Yannick Chantrana , Christophe Corpechotb, David Haddouk, et al., Groupe d'étude de l'auto-immunité (GEAI), 8eme Colloque, Anticorps anti-gp210 et anticorps anti-Sp100 dans la cirrhose biliaire primitive: une association de très mauvais pronostic, n°464 bis, juillet/août 2014
- 8: Karsten Conrad, Werner Schössler, Falk Hiepe, Marvin J. Fritzler, Book "Autoantibodies in organ Autoimmune Diseases", Volume 8, second edition – 2017
- 9: Vanderlocht J, van der Cruys M, Stals F, Bakker-Jonges L, Damoiseaux J. Multiplex autoantibody detection for autoimmune liver diseases and autoimmune gastritis. *J Immunol Methods.* 2017 Sep;448:21-25. doi: 10.1016/j.jim.2017.05.003. Epub 2017 May 16. PMID: 28522403.

**12. TEST LIMITATIONS**

1. The results obtained with this confirmatory test are dependent on the intrinsic performance of the kit and must be considered as an aid to the final diagnosis, taking into account the results obtained by a reference technique and the clinical data of the patient.
2. In case of hyper-lipemic samples, it is recommended to centrifuge it before the pipetting of the 10µl of sample, which must be done into the supernatant.



We Apply Science



IFU - Instructions for Use

LI7DIV-24/p. 10 of 12



We Apply Science





We Apply Science

